

# STYLE X<sup>®</sup> STYLE X ULTRA

STYLISH, RIGID AND  
LIGHTWEIGHT



Adjustable and fully configurable  
is the perfect chair for your lifestyle



# STYLISH, RIGID AND LIGHTWEIGHT



The Breezy Style X® and the Style X Ultra design provides a robust and rigid frame while keeping it lightweight, so you can move around comfortably for your daily activities

## STYLISH

The compact frame (80° hanger) combined with the modern design of all the Style X and Style X Ultra elements creates an attractive and active look. Available in 5 colours (navy blue, signal white, apple green, brilliant red and selenium grey).



Innovative castor housing



Compact frame: 80° hangers

## RIGID & LIGHTWEIGHT

The Style X and X Ultra chair incorporates many elements specifically though to increase rigidity. Like back tube and seat frame receivers, active seat rail fixation, two strong arm cross braces and the innovative castor house. All these parts are made of aluminium to provide strength and durability while keeping the chair lightweight.



Seat frame receiver & active campling



Aluminium axle plate with aluminium back tube receiver

# COMFORTABLE

## EASY TRANSFERS

The ergonomic release latch allows an easy lock and unlock of the footrest.

70° & 80° detachable hangers can swing outwards and inwards to facilitate transfers or shorten the chair length.

The armrests are detachable for easy user transfer, and height adjustable, with a soft armpad that provides a great user comfort.

## COMFORTABLE SEATING

The comfortable padded seat incorporates velcro straps to secure the fixation of a cushion and avoid it sliding.

Fully padded back sling with tension adjustment offers a very posture comfortable feeling for the user and allows the back to accommodate to the user back shape and comfort.

## EFFECTIVE BRAKE

The knee lever brake is very comfortable to use thanks to its big ergonomic handle, and provides a very effective brake with minimum effort.



# ADJUSTABLE

The Style X and X Ultra offers many adjustments to adapt the chair to the user measurements and needs.



## HEIGHT ADJUSTABLE BACKREST

The back height can be adjusted from 37.5 to 40 and 42.5 cm or from 40 to 42.5 and 45 cm for maximum comfort.

## SEAT DEPTH, SEAT HEIGHT AND SEAT ANGLE ADJUSTMENTS

The seat depth can be easily adjusted without tools, through the sliding cross brace from 41 to 43.5 cm, and with tools to 46 cm. The seat height can be adjusted from 38.5 cm to 51 cm. Moving the castor position on the fork, the seat angle can be changed.



## CENTRE OF GRAVITY ADJUSTMENT

Sliding the axle plate to the rear or to the front the chair's center of gravity can be moved to 2 more active positions (-5, -2.5 cm) and to 2 more passive positions (+2.5 cm, +5 cm) for additional stability.

# CONFIGURABLE

The Style X Ultra offers a wide range of options and accessories to configure the chair to the user needs.



- Reclining back
- 80° hangers
- Tool height adjustable single post armrest
- Anti-tip tubes



70° hangers



Elevating legrests



Armrest with armpad, height adjustable



Armrest with armpad, tool height and depth adjustable



- Fixed backrest
- Vented upholstery
- Height adjustable single post armrest

Castor  
solid 5"



Castor  
solid 8"



Castor  
solid 6"



Castor  
solid 8" x 2"



Castor  
solid soft 6"



Castor  
pneumatic 8" x 1"



Half folding back



Stabilizer bar



Tension adjustable back with 4 straps.  
Also available in Style X Ultra with 7 straps.

For more information about the complete range of options and accessories please check the Order Form.



	STYLE X	STYLE X ULTRA
Seat Width (cm)	38 / 40,5 / 43 / 45,5 / 48 / 52 cm	38 / 40,5 / 43 / 45,5 / 48 / 52 cm
Seat Depth (cm)	43,5 cm	41/43.5/46 cm
Seat Height (cm)	48,5 cm	38.5/41/43.5/46/48.5/51 cm (front)
Seat Angle	0° a 6 °	0° to 6°
Back Height (cm)	40 a 45 cm (48 cm with reclining backrest)	37.5 - 45 cm, 480 cm for recliner
Back Angle	10 ° (From 0 to 30° with reclining backrest)	10 ° (recliner option from 0 to 30°)
Total Length (cm)	From 91 to 106 cm (hangers 70° and 24" wheels)	from 88.5 to 106 cm
Total Width (cm)	From 56 to 70 cm	from 52.5 to 70 cm (52.5 cm 12" rear wheel)
Total Height (cm)	From 80,5 to 99 cm (6",8" front wheels) (99 cm with reclining backrest)	from 78 to 99 cm (castor 5",6",8") (99 cm recliner)
Safe Slope	From 0° to 10 °	0° to 10 °
Max. User Weight	125 kg	125 kg
Chairs Weight	From 14.7 kg (24" rear wheel)	from 13.8 kg (with 12" wheel)
Colours	Navy blue, signal white, apple green, brilliant red and selenium grey	Navy blue, signal white, apple green, brilliant red and selenium grey
Centre of Gravity Adjustment (mm)	5 / 2, 5 / 0 / -2,5 / -5 cm	5/2.5/0/-2.5/-5 cm



**Sunrise Medical S.L**

Poligono Bakiola, 41  
 48494 Arrankudiaga  
 Vizcaya, Spain  
 Tel.: +34 902 142 434  
 Fax.:+34 946 481 575

E-Mail: [Monica.garcia@sunrisemedical.es](mailto:Monica.garcia@sunrisemedical.es)  
[Patricia.delafuente@sunrisemedical.es](mailto:Patricia.delafuente@sunrisemedical.es)  
[www.SunriseMedical.eu](http://www.SunriseMedical.eu)



For further information on the full specification, options and accessories, please refer to the order form. All information is subject to change without notification. Please consult Sunrise Medical with any queries you may have. If you are visually impaired, this document can be viewed in PDF format at [www.SunriseMedical.eu](http://www.SunriseMedical.eu).

